

TRAINING AUTO-INJECTOR - USER MANUAL

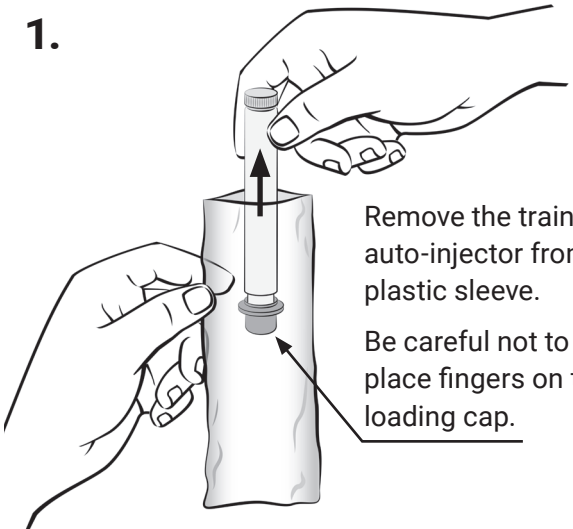
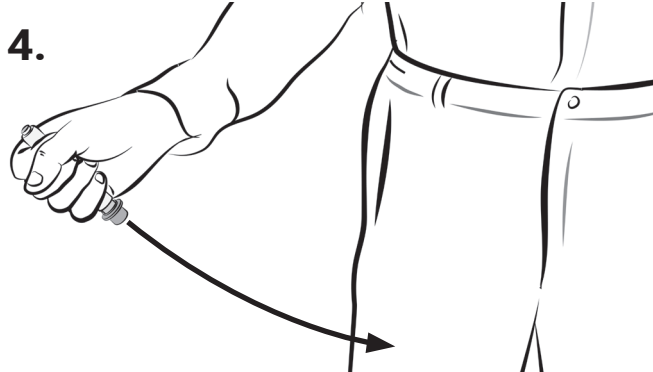
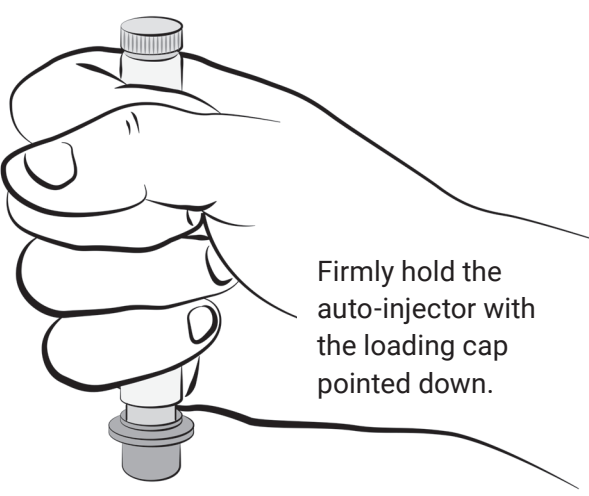
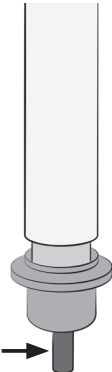
The training auto-injector simulates the action of a drug pre-filled auto-injector.

It is intended **for practice purpose only**.

It contains neither any chemicals nor a needle and does not constitute a health or safety hazard.

It is reloadable for multiple use.

How to use the training auto-injector:


1.  Remove the training auto-injector from the plastic sleeve. Be careful not to place fingers on the loading cap.	4.  Aim and firmly jab the loading cap straight down (at a 90° angle) against the mid-outer thigh. You can inject through clothing, but move all objects from in and around the patient's clothing that may interfere with the injection. Note: For people who do not have a lot of fat at the mid-outer thigh, pinch a fold of skin firmly at the injection site to provide a thicker area for administration. Hold the auto-injector firmly in place for at least 10 seconds, to mimic allowing the injection to finish. 
2.  Firmly hold the auto-injector with the loading cap pointed down.	5. Remove the training auto-injector from the mid-outer thigh and massage the injection site in a circle motion for several seconds to mimic actual use. Check that the black plastic pin (simulated needle) is out. This indicates a successful activation of the auto-injector. Note: The actual drug product includes a needle which should be bent after use and attached to the clothes of the person being administered with product, in a visible area (i.e., upper sleeve). 
3.  Pull off the safety pin with your other hand.	

How to reload the training auto-injector for later re-use:


- 1.** Hold the training auto-injector in your hand. Place your thumb and index finger on the flange of the loading cap, gently extend the cap from the injector's body and hold it stably in this position.



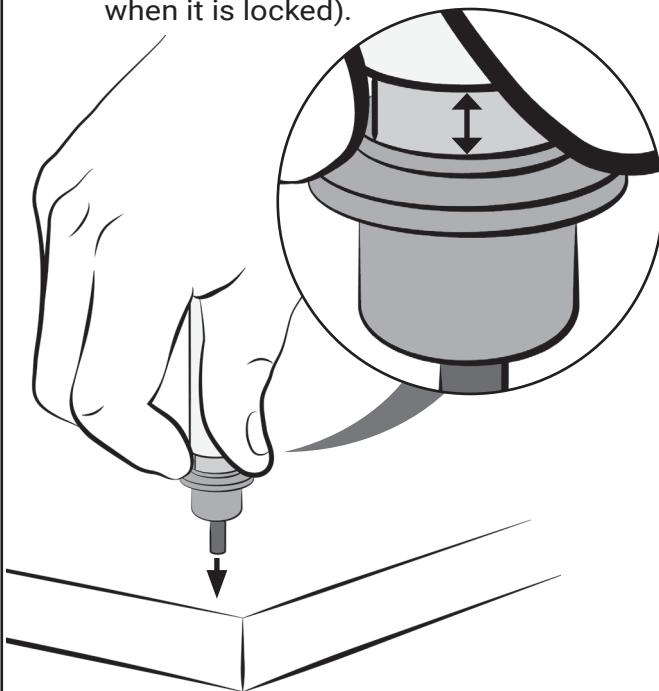
- 2.** Stably holding the loading cap fully extended from the injector's body with your thumb and index finger, press the black pin (simulated needle) perpendicularly against a flat, smooth and hard surface until the black pin is fully contracted into the cap and is locked in (you should hear a 'click' when it is locked).



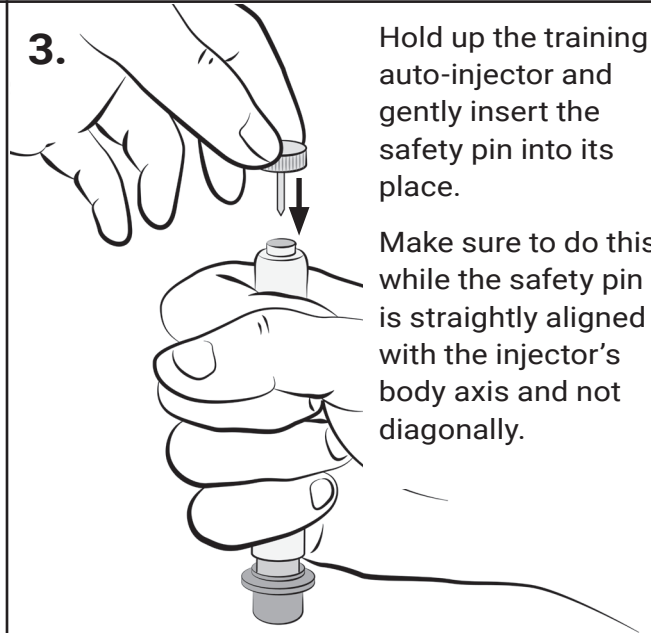- 3.** Hold up the training auto-injector and gently insert the safety pin into its place. Make sure to do this while the safety pin is straightly aligned with the injector's body axis and not diagonally.



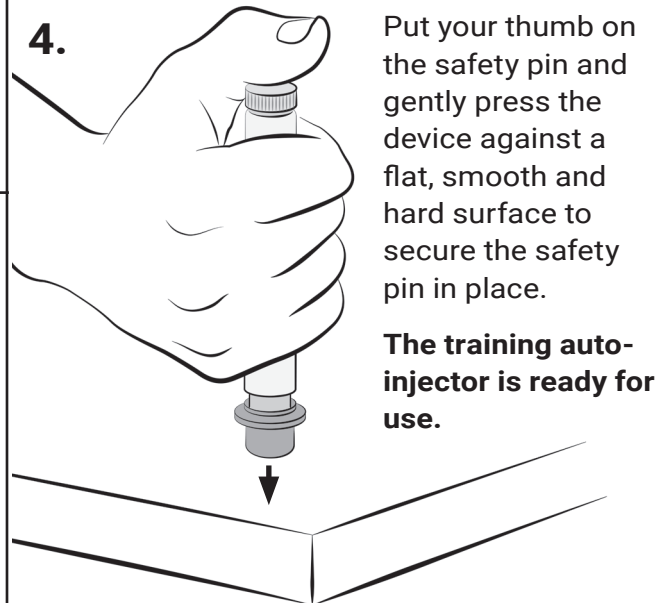- 4.** Put your thumb on the safety pin and gently press the device against a flat, smooth and hard surface to secure the safety pin in place.



The training auto-injector is ready for use.

- 5.** Put the device back into its plastic sleeve.

